

Installation instructions

# Booster seat with car restraint system



- Auto-aanpassing
- Revalidatietechniek
- Mobiliteits- en hulpmiddelen

*Onbeperkt in beweging.nl*

## Points for attention correct adjustment

It is important that the restraint system is set correctly. Make sure the connecting straps between the restraining vest and basic belt are adjusted to the correct length, not too loose and certainly not too tight. The restraint system must be comfortable for the person sitting in it.

Always use the standard safety belt for safe and legally permitted transportation.

Make sure the restraint vest is not mounted too high.

Mounting points at the level of the armpit are located at the armpit.

Mounting points at the hip are located at the hip.

For extra strength, you can choose to attach the base strap to the back of the rear seat/car seat. This way the booster seat is shut in against the back seat/car seat.

Pass the shoulder straps through the correct passage.



Make sure that the connecting straps are threaded through the ladder buckle in this way.



## Adjusting the back height

The car restraint system is mounted in the booster seat. You can adjust the back height of the combination as follows. Unlock the 7th point closure at the rear and loosen the strap (1). Then unlock the safety lock of the basic belt (2).

Raise/lower the backrest of the booster seat to the required height by means of the adjustment knob of the booster seat (A and see user manual booster seat). Then position the restraining vest at the correct height (3/4) at the front of the chair. Do not forget to use the correct shoulder strap passage.

Pull the basic belt at the back and fix the safety lock (2). Then pull the strap of the 7th point and fix it with the closure (1).

Mount the restraining vest to the basic belt with the seven side buckles. Set the desired length of the connection straps according to the measurements of the occupant. When set to the correct size, make sure that you secure the straps so that they do not slip.



Finally always use the standard safety belt for safe and legally permitted transportation of the child in the car.



## Legislation and safety

With regard to the transportation of children we advise you to consult the applicable laws and regulations.



When using the B&S Car restraint system, the use of the regular 3-point safety belt is mandatory.



The use of the B&S Car restraint system is at all times at your own risk.



Only lock the lockable side release buckles if this is really necessary.



We advise (especially when used by third parties) to coordinate with all parties involved that a person is being transported with a B&S Car restraint system.



We strongly recommend fitting the supplied safety belt cutter in the car/taxi(bus).

## Lockable side release buckles



Tighten lock buckles only when absolutely necessary! Please try after some time if it is possible to release some or all of the seven buckles.

## CE-marking



Hereby we declare that the integrated B&S Car restraint system conforms with the equivalent rules of the medical product and the guideline 93/42EEG.

## Maintenance



The integrated B&S Car restraint system can be cleaned with a light damp cloth. Do not use aggressive cleaning products and do not wash them in the washing machine!  
For the washing instructions for the covers of the Recaro Monza Nova Evo, we refer you to the manual of the booster seat.